Tracheostomie Tubus „Biesalski“ Tracheostomy cannula „Biesalski“

REF	Bezeichnung Description	Order no.	Farb / Color Code	Länge, length [mm]	Größe, size I.D. [mm]	O.D. [mm]	O.D. Schild [mm]	O.D. Kon., Con. [mm]	Winkel, angle [°]	I.C.- I.D. [mm]
TM-120900 TM-858310	Tracheostomie Tubus „Biesalski“ Tracheostomy cannula „Biesalski“	TM-120900-000040	Weiß / white	40	4	5,6	8	21	60	2,3
		TM-858310-000050								
		TM-120900-000050	Grün / green	43	5	6,5	9	21	70	3,3
		TM-858310-000050								
		TM-120900-000060	Orange / orange	47	6	7,7	10	21	70	3,9
		TM-858310-000060								
		TM-120900-000070	Braun / brown	56	7	8,6	11	21	80	5
		TM-858310-000070								
		TM-120900-000080	Rot / red	62	8	9,7	12	23	85	5,9
		TM-858310-000080								
		TM-120900-000090	Gelb / yellow	65	9	11,5	14,5	23	85	7
		TM-858310-000090								
		TM-120900-000100	Grau / grey	67	10	12,1	15	26,5	90	7,9
		TM-858310-000100								
		TM-120900-000110	Violett / violet	67	11	12,8	15	26,5	90	8
		TM-858310-000110								
		TM-120900-000120	Blau / blue	71	12	13,9	16,5	26,5	90	9,8
		TM-858310-000120								

Beschreibung / Description:

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These products each consist of a tracheostomy cannula with fixed fixation flange, speaking valve, cough cap, two inner cannulas and a neck band.

Biesalski cannulas are used in spontaneously breathing patients with stable stomata and do not require an insertion aid.

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Beim Typ Biesalski wird die Innenkanüle mittels Bajonett Verschluss fixiert, der sich jeweils an Konnektor, Sprechventil und Hustenkappe befindet. *The tracheostomy cannula can optionally be used with or without an inner cannula. The inner cannula is mainly intended for patients who, based on their underlying disease, exhibit increased secretion formation. The inner lumen of the cannula can become constricted by secretions (mucosa, but also blood or other body fluids). In this case only the inner cannula needs to be removed and replaced by a new one. The tracheostomy cannula itself does not need to be removed.*

In Biesalski cannulas, the inner cannula is fixed by means of a bayonet fastener, which is located at the connector, phonation valve or cough cap, respectively.

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The correct selection of the cannula size depends on the size of the available tracheostoma, the tracheal diameter and the underlying disease of the patient, and is primarily the responsibility of the treating physician. As a rule, in order to secure optimal lung aeration (and concurrently keeping airway resistance as low as possible), the maximum size tube is selected that can be incorporated, based on the anatomical proportions of the trachea and the tracheostoma of the patient.

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Prior to use, the tracheostomy set must be examined for completeness and functionality.

Before intubation, the lumen should be examined for patency.

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Insertion of the cannula is facilitated when the head of the patient is extended and the tracheostoma is widened with the thumb and index finger of the other hand.

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PLEASE NOTE: Please take care that the cannula shaft does not exert any pressure on the tracheal rings and is positioned in the tracheostoma free of any tension.

4. Fixieren der Halteplatte am Hals des Patienten / Fastening the fixation flange to the patient's neck

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**Tracheostomie Tubus „Biesalski“
Tracheostomy cannula „Biesalski“**

After insertion of the cannula, the entire system must be fastened onto the patient's neck in order to prevent the cannula from slipping out. The ends of the neck band are led through the respective fastening slits in the fixation flange and are subsequently fastened. To prevent irritation of the skin around the neck where the fixation flange is positioned – especially in the case of an oozing tracheostoma – inserting a commercially available tracheostomy dressing between skin and fixation flange has been proven effective.

5. Einführen der Innenkanüle (bei Bedarf) / Insertion of the cannula (when required)

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The inner cannula should be fully inserted without exerting too much pressure onto the tracheostomy cannula.

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The used tracheostomy cannula should be disposed according to the nationally applicable directives for waste disposal.

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Store in a dry place and shielded from direct or indirect lighting, humidity and heat sources.

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Each tracheostomy set is sterile and is supplied individually packaged in a wrapper.

The sterility of the tracheostomy set and its components is only assured as long as the packaging unit is unopened and intact, however no longer than indicated on the label.

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Suitable cleaning and sterilisation methods and instructions for reprocessing according to the original specifications are not available for these products. Re-use of disposable products can lead to decreased performance or to loss of functionality, or to exposure to pathogens; validated verification procedures are not available.

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