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Uretersplint, standard

Description

Ureter splints have a closed end or central opening for coaxial application. They are either perforated throughout the entire length or only in the pigtail area and marked every cm. X-ray. Sets are supplied with a guide wire, some with a pusher.

Length between the curves: 22 / 24 / 26 / 28 cm

Ch.	4,8	6,0	7,0	8,0
Max. diameter of guide wire [inch]	0,032	0,035	0,038	0,038

Indications

Standard splints are intended for use as routine ureter stents. Application has to be done by a physician or suitably qualified medical personnel.

The ureter splint is utilised to lead fluid from the renal pelvis into the urinary bladder.

- universal ureter blockages and stenoses
- supportive measures when treating renal or ureteral calculi
- trauma

Contraindications

Insurmountable ureteric obstruction.

Preparation of use

Length and diameter must be individually suitable to the patient. It is important that the outer diameter of the splint is compatible to the inner diameter of the endoscope. Remove the splint set from their sterile pouches using an aseptic technique. Do not use kinked guide wires.

Potential complications

Infections by improper use, blockade of the stent due to encrustation, perforation and irritation of bladder and ureter.

In order to assure the flow in the stent the patient should be routinely monitored.

Application

- Introduce the flexible guide wire via the working channel of the optical instrument.
- Localise and probe the ureter and position the guide wire end in the renal pelvis.
- Insert the ureter splint using the already implanted guide wire.
- Utilise the pusher to bring forward the ureter splint via the lying guide wire to the renal pelvis. The position is reached as soon as the distal pigtail of the ureter splint completely deploys.
- Keep the pusher in its position and withdraw the guide wire. After a position check remove the guide wire and the pusher.

Supply

Every set is supplied sterile in an individual pouch.

The sterility of the set and its components is guaranteed as long as the packaging unit is unopened and intact, but no longer than stated on the label.

Storage

The product should be stored in a dry place and protected against direct an indirect light, moisture and heat sources.

Single-use product

These products are for single use only. Reprocessing may result in degraded performance or loss of functionality. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these devices. This product is not designed to be cleaned, disinfected or sterilized.

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